

Disposal of General X-ray Tube Head

General X-ray equipment may be disposed of using the following guidelines.

- Remove power source so that it is impossible to power on the equipment. I.e., Remove any batteries and/or power cords.
- Remove or deface any radiation labels and/or symbols.
- Drain any oil, for instance oil used to cool the tube. The oil will need to be disposed of properly. Contact the waste management facility in your area for proper disposal methods.

For 3rd party removal you can find X-ray Sales and Service Providers on our website

<https://www.healthy.arkansas.gov/programs-services/topics/x-ray-vendors>

Arkansas Department of Health
Radiation Control
4815 W. Markham, Slot 30
Little Rock, AR 72205

Phone (501)661-2378
Fax (501)661-2301